

Care Guide | Servox® Digital XL Electro Larynx Speech Aid

BATTERIES

Your Servox® digital XL comes with 8 AAA batteries, these should be fully charged prior to use. Even when not being used, the batteries must be kept in a charged state to prolong their life. If required, non-rechargeable AAA batteries may be used, but do not mix brands or they could leak.

BATTERY MODULE

Before you can insert the batteries they must first be loaded into the battery module, which is designed to hold 4 AAA batteries. Observe the correct polarities as marked on the tail end of the module.

INSERTING THE MODULE

The correct alignment of the battery module is as shown below. A raised rib and an indent on the end of the module lined up with the top of the Servox® digital XL makes the process easy.



BATTERY END CAP

Once the battery model has been correctly inserted, screw on the battery end cap, taking care not to over tighten.

SAFETY CORD

The Servox® digital XL in many cases is the only means of communication. To avoid accidental dropping or loss, it is advisable to use the safety cord provided.

WATER

Avoid getting the Servox® digital XL wet as water and electronics do not go well together. The O-ring fitted under the head cap is not there to render the Servox® digital XL waterproof, it is to prevent over tightening.

BATTERY CHARGER AND TRAVEL ADAPTER

The new Servox comes with a Universal Travel Adapter. It is in the white box with the orange stripe. Inside that you will find a little black box with two plugs. Select the one with the Australian points and plug this into the VARTA Battery Charger (see the attached image). You may need to swivel the 240V points to fit the shape of the power point. This then fits into the wall as you can see in the last attached image.



Proudly distributed by