REF 1700302

Rx Only

REF 2100394-0002/2022.05

**Instructions for use**  
**S120**

Please read carefully and keep  
this copy for further use

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### 1. SYMBOLS USED

	<b>CE Marking</b> with the number of the notified body.
Rx Only	<b>Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.</b>
	<b>CAUTION!</b> Refer to the accompanying documents
	<b>Consult the accompanying documents</b>
	<b>Materials to be recycled</b> The disposal and/or recycling of materials must be performed in accordance with the directives and the legislation in force.
	<b>Sterilizable in autoclave up to the specified temperature</b>
	<b>Cleaning in machine authorized</b>
	<b>Electrical or electronic materials to be recycled.</b>
	<b>Manufacturer</b>

### 2. INTENDED USE

The S120 is an active medical device intended to convert electrical energy to mechanically drive shaver blades and bur for cutting soft tissue and bone.

### 3. INDICATION FOR USE

The S120 is intended for professional use and it has been designed for shaping bones and for the resection of soft and hard tissue in the fields of head & neck/ENT (rhinology, laryngology).

### 4. CONTRAINDICATION

None currently known.

### 5. PRECAUTIONS - WARNINGS

#### 5.1. Warnings, Precautions for use

For additional information, please contact Bien-Air Surgery SA at the address indicated on the first page of this document.

#### CAUTION

**Use adequate irrigation and avoid excessive pressure on the tool. The use of a tool with insufficient irrigation and with excessive pressure may cause an inordinate amount of heat buildup resulting in a thermal injury to tissue. See tools instruction for use for further information.**

#### CAUTION

**Do not use this device in the presence of a flammable anesthetics. Avoid potential ignition or explosion of gases.**

The device and its accessories should be used only by duly trained and competent medical personnel, in particular in compliance with the legal provisions in force regarding occupational safety, health and accident prevention measures, and the present user manual. According to these measures, the user has the following obligations:

- To only use devices that are in perfect working condition. In the event of irregular operation, excessive vibrations, abnormal overheating or other signs suggesting malfunctioning of the device, work must be suspended immediately. In this case, contact a repair center approved by the legal manufacturer.
- Make sure that the device is used only for the purpose for which it is intended. Protect yourself, patients and third parties from all danger and avoid contamination by the product.

The device and its accessories are designed solely for medical treatment. Any use that does not comply with its intended use is unauthorized and may prove dangerous. This medical device complies with European legal provisions in force.

Do not modify this equipment without authorization from the manufacturer. If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of the equipment.

Use only original maintenance products, accessories and/or spare parts approved by the legal manufacturer. The use of other products, accessories or parts could void the guarantee and/or endanger the patient or the operator.

#### 5.2. Environmental protection and indications for device disposal

This equipment must be recycled. The disposal and/or recycling of materials must be performed in accordance with the directives and the legislation in force.

Electrical and electronic equipment may contain dangerous substances which constitute health and environmental hazards.

The user can return the device to their distributor or directly call a firm certified to treat and recover this type of equipment (European directive 2012/19/EU).

## 6. TECHNICAL DATA

### Environmental conditions:

<b>Work</b>	Temperature:	+10°C to +30°C (+50°F to +86°F)
	Relative humidity:	20% to 80%, including condensation
	Atmospheric pressure:	700 hPa to 1060 hPa
<b>Transport</b>	Temperature:	-25°C to +70°C (-13°F to +158°F)
	Relative humidity:	10% to 100%, including condensation
	Atmospheric pressure:	600 hPa to 1060 hPa
<b>Storage</b>	Temperature:	0°C to +40°C (+32°F to +104°F)
	Relative humidity:	10% to 90%, including condensation
	Atmospheric pressure:	600 hPa to 1060 hPa

### Type:

Direct current, brushless and sensorless.

### Maximum Rotation speed:

	Continuous rotation [rpm]	Oscillating mode [rpm]
ORIGO	300 to 12'000	500 to 5'000
OSSEODUO	600 to 12'000	
OSSEOUNO	500 to 8'000	

### Noise level:

As per ISO 11498: < 62 dBA at 45 cm.

### Size / Weight:

Diameter 18 mm, length 140 mm, angle 15°/198 g without cable and shaver blade/bur.

### Period of operation:

#### CAUTION

To avoid overheating leading to burns (temperature between 41°C (106°F) and 48°C (118°F) on the outer surface of the micromotor), the following rules must be respected:

1. Limit the maximum speed of rotation based on the intended use.
2. Sufficient irrigation is recommended.
3. Duty cycle:

For normal operating temperature room (typically 20°C) the S120 is rated as below:

	ORIGO
S120	10s ON ; 5s OFF

For an operating room at 30°C, the S120 is rated as below:

	OSSEODUO OSSEOUNO	ORIGO
S120	2 min. ON ; 20 min. OFF	30 sec. ON ; 2 min. OFF

### Medical class:

Ila as per 93/42 EEC directive.

### Motor cable:

Push-pull type connector at ORIGO, OSSEODUO and OSSEOUNO end. Cable length: 295cm ±5cm.

### Tools:

Single-use and reusable straight and curved shaver blades, bur blades.

### Compatibility:

S120 is compatible with ORIGO, OSSEODUO and OSSEOUNO system.

For further information concerning microsurgery instruments and their accessories, please contact your Bien-Air Surgery local distributor or consult our website [www.bienair.com](http://www.bienair.com).

## 7. USE

The S120 is designed to drive various types of Shaver surgical blades. It is connected to the control unit by a specific cable.

### 7.1. Irrigation

1. Connect the irrigation line to the S120 up to the thrust stop.
2. Attach the irrigation line to the motor cable using the irrigation clips provided.

### 7.2. Suction

1. Connect the suction hose with inner diameter of 4 to 8 mm to the S120.
2. Insert the suction hose into the apertures of the mounting clips already installed on the motor cable.

### 7.3. Surgical shaver blades and burs

The blades and burs for the S120 are sterile, disposable instruments or reusable and sterilizable instruments.

The shaver blades and burs are used exclusively with the S120. There are various blades and burs depending on the planned operation. Refer to the documentation for the shaver blades and burs.

Before using the S120, operate it at a moderate speed without a blade for a few seconds, so as to spread the lubricant and remove any excess.

#### CAUTION

Never insert a shaver blade/bur onto a rotating S120.

### 7.4. Shaver blade or bur insertion and removal on the S120

1. Gently insert the blade/bur in the S120 up to the thrust stop; the clamping ring should retract and then return to its initial position. If the blade/bur is not correctly in place (joint still visible), remove it and start insertion again.
2. Pull the blade/bur slightly to check that it is correctly attached.

3. Pull the clamping ring to the rear and remove the blade/bur.

NOTE: The blade/bur consists of two separate parts. If the internal part remains held in the S120, pull on this component axially to withdraw it.

#### 360° orientation system:

1. Turn the orientation crown to choose the correct shaver blade position.
2. The auto-blocking system maintains the shaver blade in its position.

#### 7.5. Motor cable connection

Before connecting the specific motor cable, check the cleanliness of the rear of the S120 and the cable plug.

1. Present the S120 in correct alignment (align the cable reference with the reference on the motor).
2. Insert the plastic connector in position.
3. Tighten the connector nut as far as possible.
4. Before use, run the S120 at moderate speed for a few seconds to distribute and remove excess lubricant.

#### 8. CLEANING / INSPECTION / STERILIZATION

##### CAUTION

Do not place the S120 in an ultrasonic bath.

##### CAUTION

Never submerge the S120 in disinfection solutions or physiological saltwater solutions (NaCl solution), as prolonged contact may cause corrosion.

##### CAUTION

To avoid water staying between the cable and the S120 always hold/place the S120 with the front end down (minimum 45°) during the entire cleaning process.

##### CAUTION

**Chemical disinfection of the S120 is not recommended due to possible negative effects on the lifetime of the devices and possible residues of disinfectants.**

##### CAUTION

**This device is delivered "non-sterile".**

Before use please comply with these instructions.

##### Precautions for use:

Hospital procedures must be followed. Hospital personnel working with contaminated or potentially contaminated medical instruments must comply with universal precautions. Pointed and sharp instruments should be handled with great care.

##### Agents required for cleaning:

###### • Detergents

The cleaning of the micromotor has been validated using an enzymatic pH neutral detergent (Steris Prolystica® 2X Concentrate Enzymatic Presoak and Cleaner). An alkaline detergent might reduce the lifetime of the device.

Detergents should be used at the concentration, temperature and for the duration recommended by the detergent's manufacturer.

To remove physiological liquid inside the instrument or the spray tube, use "Aquacare" from Bien-Air Surgery SA.

##### CAUTION

**Do not use detergents that are corrosive or contain chlorine, acetone or bleach, aldehydic products or alcohols.**

###### • Lubricant:

Only use "Lubrifiuid" REF 1600064 from Bien-Air Surgery SA.

###### • Brush / cleaning gun

Brushes should be non-aggressive to avoid damaging the device. Preferably use nylon brushes

with flexible or soft bristles. Use the cleaning gun with the appropriate nozzles for the various ducts.

Before use, please comply with the present section:

- Clean, lubricate and sterilize the S120 before first use.
- Clean, lubricate and sterilize the S120 before each further use.
- After each use, clean and sterilize the S120 as quickly as possible.

If you have any additional questions regarding reprocessing procedures, instructions for reusable devices etc., please contact Bien-Air Surgery. Contact information can be found on the first page of this instructions.

#### 8.1. Point of Use cleaning:

##### CAUTION

**Initial cleaning should be performed at the point of use and as soon as possible after the completion of the surgical procedure.**

##### CAUTION

**Point of use cleaning must be followed by Manual or automatic cleaning.**

This operation is important in order to facilitate subsequent cleaning stages (it prevents dirt from drying and sticking to the equipment).

As soon as you have finished using the S120, proceed as follows:

1. Disconnect the cable from the control unit.
2. Do not disconnect the cable from the S120.
3. Disconnect the irrigation and suction lines from the S120.
4. Separate the surgical shaver blade or bur from the S120 and treat it in accordance with its instructions or in accordance with the local regulations in force concerning the disposal of contaminated wastes.
5. Rub the outside surfaces of the S120 with non-woven towelettes (pre-soaked in water).

- Spray the irrigation channel in the direction it is used with "Aquacare" or inject tap water with a syringe.
- Spray the suction path from the blade side of the instrument with "Aquacare" or inject tap water with a syringe.
- Ensure that the S120 do not dry before manual or automatic cleaning by wrapping all parts in non-woven towelettes (pre-soaked in water).
- Manual or automatic cleaning must be done max 1h after the point of use.

## 8.2. Manual cleaning

Thoroughly rinse the S120 and its cable under running tap water (cold) as follows :

- Place the S120 under running tap water (tip upwards) and rinse the inside (FIG.1). Block the suction channel at least 3 times with a finger for 5 seconds so the inside fills with water (FIG.2). Then turn it to let the water flow out through the distal opening.
- Keeping your finger on the suction channel (tip upwards) and under running tap water, brush the inner distal section (blade's connection side) using a Ø10-12 mm soft nylon bristles brush for at least 10 seconds. Use back-and-forth and rotating movements and make sure to push the brush up to the end.
- Hold the S120 with the distal part downwards and rinse through the suction channel for at least 10 seconds (FIG.3).
- Brush the suction channel using a Ø4-5 mm soft nylon bristles brush for at least 10 seconds. Use back-and-forth and rotating movements and make sure to push the brush through the complete length.
- Rinse the outside of the S120 including the first 10 cm of the cable. While rinsing, make sure to never direct tap water on the 3 ventilation holes (FIG.4). Brush with soft nylon bristles for at least 30 seconds on the outside surfaces, continue brushing until soil is no longer visible. Then turn

the rotation wheel at least 3 turns in each direction and press the blade release button at least 3 times. Brush the distal portion of the handpiece at the crown for at least 20 seconds. Finally, brush the junction between the cable and the handpiece for at least 20 seconds.

- Rinse the entire cable and rub with wet non-woven wipes, moving back and forth, for at least 30 seconds or until soil is no longer visible.

Thoroughly wash the S120 and its cable with an enzymatic neutral pH detergent (like Steris Prolystica® 2X Concentrate Enzymatic Presoak and Cleaner) solution as follows:

- Keeping your finger on the suction channel (tip upwards), pour detergent solution into the S120 so the inside fills up. Stay in this position for at least 30 seconds. Brush the inner distal section (blade's connection side) using a Ø10-12 mm soft nylon bristles brush soaked with detergent. Use back-and-forth and rotating movements and make sure to push the brush up to the end. Brush for at least 20 seconds, continue brushing until soil is no longer visible. If necessary, refill the device with detergent solution during the brushing procedure, at the end let it flow out through the distal opening.
- Brush the suction line using a Ø4-5 mm soft nylon bristles brush soaked with detergent. Use back-and-forth and rotating movements and make sure to push the brush through the complete length. Brush for at least 20 seconds, continue brushing until soil is no longer visible.
- Pour detergent solution over the outside surfaces of the device including the first 10 cm of the cable, making sure to cover all surfaces. While pouring, make sure to never direct detergent on the 3 ventilation holes (FIG.4).
- Brush the outside of the S120 (especially in the corner, edges and openings), including the first 10 cm of the cable, using a nylon soft bristles brush soaked with detergent. While brushing

around the distal section, turn the rotation wheel at least 3 full turns in each direction and press the blade release button at least 3 times. Brush for at least 30 seconds, continue brushing until soil is no longer visible. Brush the distal portion of the handpiece at the crown for at least 20 seconds. Finally, brush the junction between the cable and the handpiece for at least 20 seconds.

- Rub the entire cable with detergent impregnated non-woven wipes, moving back and forth for at least 30 seconds.

Thoroughly rinse the S120 and its cable under running tap water (cold) as follows:

- Place the S120 under running tap water (tip upwards) and rinse the inside (FIG.1). Block the suction channel at least 3 times with a finger for 5 seconds so the inside of the device fills with water (FIG.2). Then turn it to let the water flow out through the distal opening.
- Hold the S120 with the distal part downwards and rinse through the suction channel for at least 20 seconds (FIG.3). During rinsing, turn the rotation wheel at least 3 full turns in each direction and press the blade release button at least 3 times.
- Holding the S120 with the tip downwards, spray in the irrigation connector with "Aquacare" or by injecting water with a syringe during at least 2 seconds.
- Holding the S120 with the tip downwards, rinse the outside surfaces for at least 20 seconds. Then, rinse the entire cable for at least 20 seconds. While rinsing, make sure to never direct tap water on the 3 ventilation holes (FIG.4).

Drying:

- Dry the outside of the S120 including its cable using a non-linting wipe.

17. Dry the inside of the S120 by holding it with the distal part downwards and applying pressurized air (air gun or canned air) through the suction connection and the irrigation connection.
18. If the S120 is not immediately sterilized, perform dynamic drying under ventilation, at about ~90°C (194°F), for a minimum of 25 minutes.

**Or**

### 8.3. Automatic cleaning

#### Pre-cleaning:

1. Place the S120 under running tap water (tip upwards) and rinse with cold water for at least 30 seconds to clean the inside. Block the suction channel at least 3 times with a finger. Follow with a 10 second rinse with the tip downwards through the suction channel (FIG.1 to FIG.3).
2. Wash the S120 by pouring detergent solution (like Steris Polystica® 2X Concentrate Enzymatic Presoak and Cleaner) into the distal opening (tip upwards).
3. Brush for at least 10 seconds the inner distal section of the S120 (blade's connection side) using a Ø10-12 mm nylon soft bristles brush soaked with detergent. Use back-and-forth and rotating movements and make sure to push the brush up to the end. Then turn the rotation wheel at least 3 turns in each direction and press the blade release button at least 3 times. Brush again for at least 10 seconds, continue brushing until soil is no longer visible.
4. Brush the suction channel using a Ø4-5 mm nylon soft bristles brush soaked with detergent. Use back-and-forth and rotating movements and make sure to push the brush through the complete length. Brush for at least 20 seconds, continue brushing until soil is no longer visible.
5. Brush the outside of the S120 (especially in the corner, edges and openings) using a nylon soft bristles brush soaked with detergent. While brushing around the distal section, turn the rotation wheel at least 3 full turns in each

- direction and press the blade release button at least 3 times. Brush for at least 15 seconds, continue brushing until soil is no longer visible.
6. Rub the entire cable with detergent impregnated non-woven wipes, moving back and forth.
  7. Rinse the inner of the S120 with running tap water (cold) from the distal opening (tip upwards) for at least 20 seconds. Block the suction channel at least 3 times with a finger. Follow with a 10 second rinse with the tip downwards through the suction port (FIG.1 to FIG.3). During rinsing, turn the rotation wheel at least 3 full turns in each direction and press the blade release button at least 3 times.
  8. Rinse the outside of the S120 and the entire cable with running tap water for at least 20 seconds.
  9. Holding the S120 with the tip downwards, spray in the irrigation connector with "Aquacare" or by injecting water with a syringe during at least 2 seconds.

Pre-cleaning must be directly followed by final automatic cleaning.

#### Automatic cleaning:

Place the S120 and its cable in the appropriate washer/disinfecter basket and treat via a standard instrument washer/disinfecter cycle (place the S120 with the tip downwards). Only use a validated washer / disinfecter.

**This product has been validated with an automated cycle consisting of the following stages:**

#### Pre-wash:

Cold tap water (<45°C / 113°F) for minimum 2 minutes.

#### Washing:

Hot tap water, 50°C to 60°C (122°F to 140°F) with enzymatic pH neutral detergent (like Steris Polystica® 2X Concentrate Enzymatic Presoak and Cleaner), minimum 5 minutes.

#### Neutralization:

Cold tap water (<45°C / 113°F) for minimum 2 minutes .

#### Rinsing:

Cold critical water acc to AAMI TIR34 (<45°C / 113°F) for minimum 2 minutes.

#### Thermal disinfection by rinsing:

Hot critical water acc to AAMI TIR34 (90°C / 194°F) for minimum 5 minutes.

Operator is responsible for the implemented value A0, according A0 concept described in EN ISO 15883 (for example, A0 600 90°C (194°F)/1 min.).

#### Ventilated dynamic drying:

70°C (158°F), for minimum 22 minutes.

### 8.4. Inspection, lubrication and testing

Carefully inspect each part to make sure that all visible contamination has been eliminated. Check in particular that the ducts are clear. Where there is contamination, repeat the cleaning process.

After each cleaning operation and before each sterilization, lubricate the instrument with "Lubrifluid" from Bien-Air Surgery SA as follows:

To absorb any excess lubricant, apply a cloth over the instrument's apertures. Insert the tapered end of the "Lubrifluid" spray in the tip of the S120 and squirt for approx. 0.5 seconds..

Then leave the S120 to rest upright with the tip downwards for at least 5 minutes. Check the freedom of movement of attachment ring. To do this, push it several times to full stop and release. It must always return to the starting position.

Check the freedom of movement of the orientation crown (in both directions of rotation).

#### Packing for sterilization:

Separate packing: Immediately insert the S120 with its connected cable in individual wrapping, such as a paper/plastic pouch or sterilization wrap for steam sterilization.

Make sure that the cable does not touch the sides of the sterilizer.

In the USA, FDA approved sterilization wraps or pouches must be used.

### 8.5. Sterilization

Sterilization by steam/moist heat is the recommended method for the S120 and its cable. The following table of sterilization parameters, using a B-type cycle with Pre-Vac, is recommended by Bien-Air Surgery SA to provide a sterility assurance level (SAL) of  $10^{-6}$ .

Only legally marketed, FDA cleared sterilizers, sterilization wrap/pouches, biological indicators etc. should be used by the end-user for packaging terminally sterilized devices.

Temperature	132°C (270°F)	134°C <sup>2</sup> (273°F)	134°C <sup>2</sup> (273°F)	135°C (275°F)
Time	4 Min.	3 Min.	18 Min. <sup>1</sup>	3 Min. <sup>1</sup>
Minimum drying time	30 Min <sup>3</sup>			

<sup>1</sup> Parameters recommended by the World Health Organization for treating instruments in the event of contamination by Non-Conventional Transmissible Agents (NCTA).

<sup>2</sup> Not for users in US healthcare facilities.

<sup>3</sup> Refer to sterilizer manufacturer recommendations for drying times per load configuration.

#### Only for users in US healthcare facilities:

Gravity steam sterilization for the S120 and its cable using sterilization pouches is alternatively possible by using the recommended parameters shown below :

- 121°C (250°F) ; 30 minutes ; Minimum drying time of 45 minutes.

### CAUTION

Do not exceed a temperature of 138°C (280°F).

### CAUTION

Never rinse instruments in cold water to cool them.

### 8.6. Storage

The legal manufacturer strongly advises storing only sterilized devices to reduce the risk of corrosion.

#### Ambient conditions for storage after sterilization:

- Store the equipment in a clean, dry place at an ambient temperature (See Chap. 6 for further details).
- Do not expose the equipment to direct sunlight.
- Do not expose the equipment to permanent X-ray irradiation.
- Do not store the equipment in places that could be subject to liquid splashes.
- Do not store the equipment in the following ambient conditions: Dust, Saline or sulfurous atmosphere
- Do not store the equipment in a location where there is a risk of the release of flammable gases.

#### Shelf life of sterilized instruments:

The shelf life of stored sterilized instruments depends on the type of packaging used and the storage conditions (refer to the DIN 58953 standard, section 9, or existing local regulations).

### 9. MAINTENANCE

No S120 components are to be changed by the user. Never disassemble the micromotor.

For all servicing and repairs, we recommend that you contact your dealer or Bien-Air surgery directly.

The legal manufacturer invites users to have their dynamic instruments checked or serviced once to twice a year depending on usage frequency.

### Hygiene

For the safety of the repair center's personnel, the instrument should be fully cleaned and sterilized before being returned for repair. If that proves

impossible, for example because disinfection or sterilization would make the instrument completely unusable, clean the instrument as carefully as possible and mark it accordingly to indicate that it has not been decontaminated.

### 10. MALFUNCTIONS AND ERRORS

Use the table below to solve problems encountered. If the problem cannot be solved, stop using the product and contact a repair center approved by the legal manufacturer.

Problem	Solution
The S120 is not working	Make sure that the cable is properly connected to the S120 and the control unit. Make sure that the control unit is turned on and S120 is selected. Check the condition of the S120 cable (broken contact or severed cable).
Impossible to insert the shaver blade or bur	Check the condition of the blade/bur. Without a blade attached, check the freedom of movement of the clamping ring (See Chap. 7.4 Blade or bur insertion and removal on the S120). - To do so, push it up to the thrust stop and release. It should return to its initial position - Check that there is no part or other body retained inside

Problem	Solution
The clamping ring is not free to move.	Check the external and internal cleanliness of the clamping system. Lubricate with "Lubrifiuid" before sterilization
The 360° orientation system does not move freely or the auto blocking system of the blade is failing.	Check the external and interior cleanliness of the clamping ring and orientation system. Lubricate with "Lubrifiuid" before sterilization.
The blade or the surgical shaver bur is inserted, but it does not rotate.	Remove the blade and check that the drive is not broken. Check that the blade is correctly in position.
The suction pump operates, but there is little or no suction.	Check that the hose is not clogged or kinked. See the blade operating instructions. Clean the suction channel of the S120 with a soft-bristle brush.
The irrigation pump operates, but the irrigation fluid does not flow.	Check that the irrigation line is properly connected and that the flow rate setting is sufficient. Check that the blade is not clogged. Check the integrity of the O-Ring. Check that the S120 duct is not clogged. This should be done without a shaver blade/bur.

## 11. GENERAL TERMS OF GUARANTEE

### 11.1. General Information

Bien-Air Surgery SA endeavors to provide its customers with products and devices of impeccable quality which are guaranteed, within the limits of the present general terms and any specific agreements signed, against any operating fault, material or manufacturing defect.

The guarantee period is 12 months from the date of invoice.

In general, the guarantee does not exempt the customer from the obligation to obtain information from Bien-Air Surgery SA in case of doubt and in particular when the product is used in conditions not explicitly provided for originally.

The buyer is obliged to check the goods received within 8 days following their receipt. If the goods are not checked within the aforementioned period, the customer shall be deemed to have accepted the goods, barring hidden defects. The defect notice must be received in writing by Bien-Air Surgery SA within the aforementioned period and must contain the customer's name, the date of purchase, the product reference and serial number.

In the event of claims, Bien-Air Surgery SA or its authorized representative shall perform product repair or replacement free of charge, after analyzing the justification for the claim.

All other claims of whatsoever kind, and in particular claims for damages, are excluded.

Bien-Air Surgery SA shall not be held responsible for damage or injury and the consequences thereof, resulting in particular from:

- excessive wear,
- inappropriate use,
- failure to comply with operating instructions, assembly instructions or maintenance instructions,
- exceptional environmental, chemical, electrical or electrolytic influences,
- faulty air or water seals or electrical connections.

In any case the guarantee becomes null and void in the event of inappropriate servicing, use of non-recommended parts, accessories or consumables, or modifications to the product carried out by third parties not authorized by Bien-Air Surgery SA.

In case of dispute as to whether or not the defect exists, it shall be incumbent on the customer to prove the existence of the defect.

Guarantee claims shall be taken into consideration only upon presentation, with the product, of a copy of the invoice or delivery slip on which should appear clearly the date of purchase and the product reference and serial number.

### 11.2. Governing law

Swiss domestic law (Code of obligations) shall apply in addition to the general terms and conditions and particular agreements between the customer and Bien-Air Surgery SA.

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