




Danis Stent Implantation Quick Guide

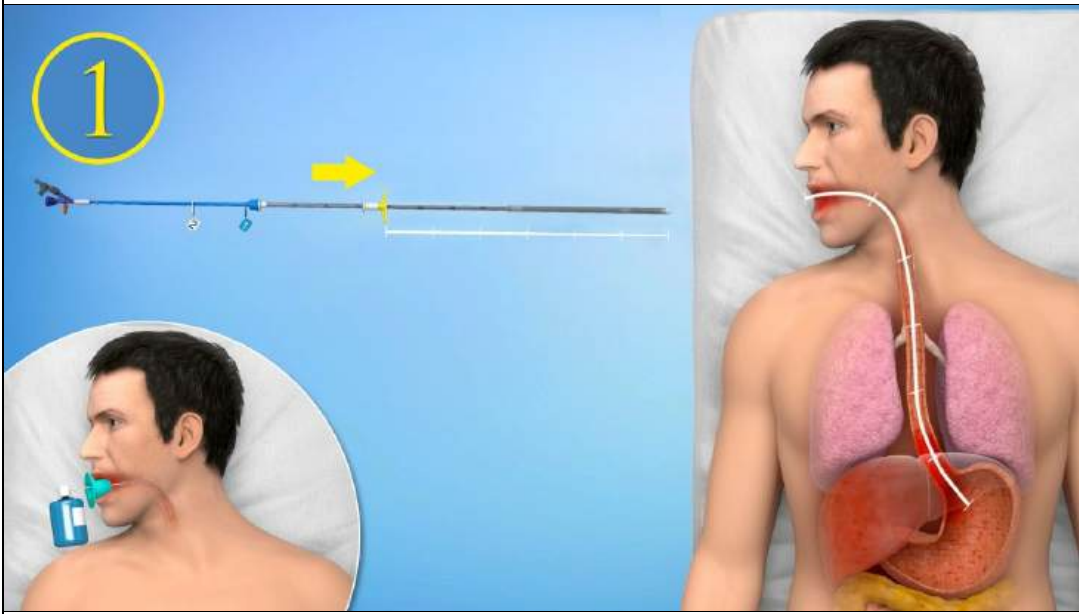


1 Measure the distance and set the yellow plate.

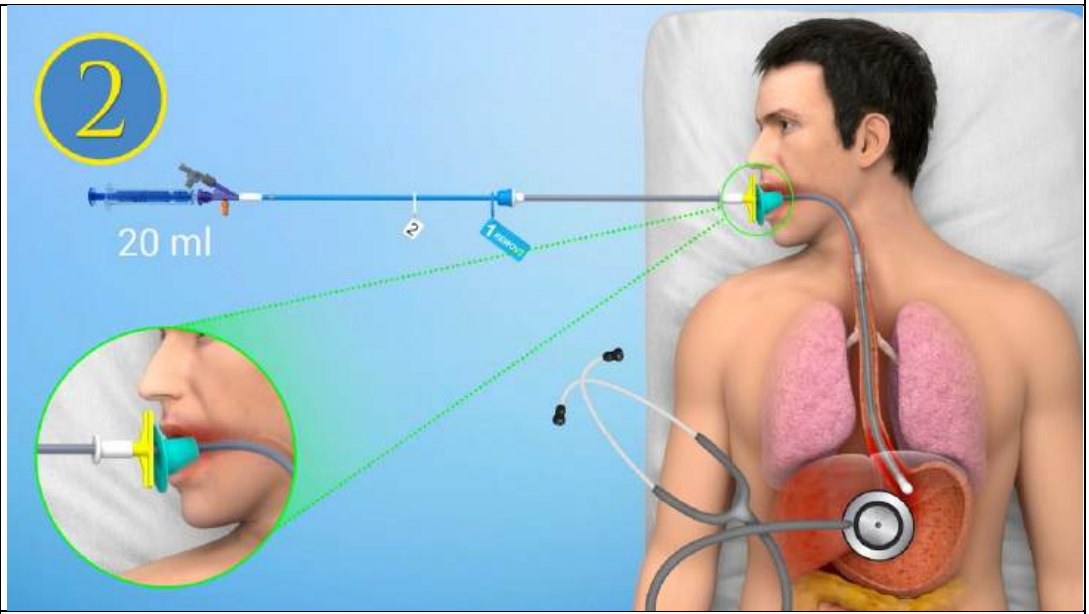


2 Introduce the delivery system and verify position.

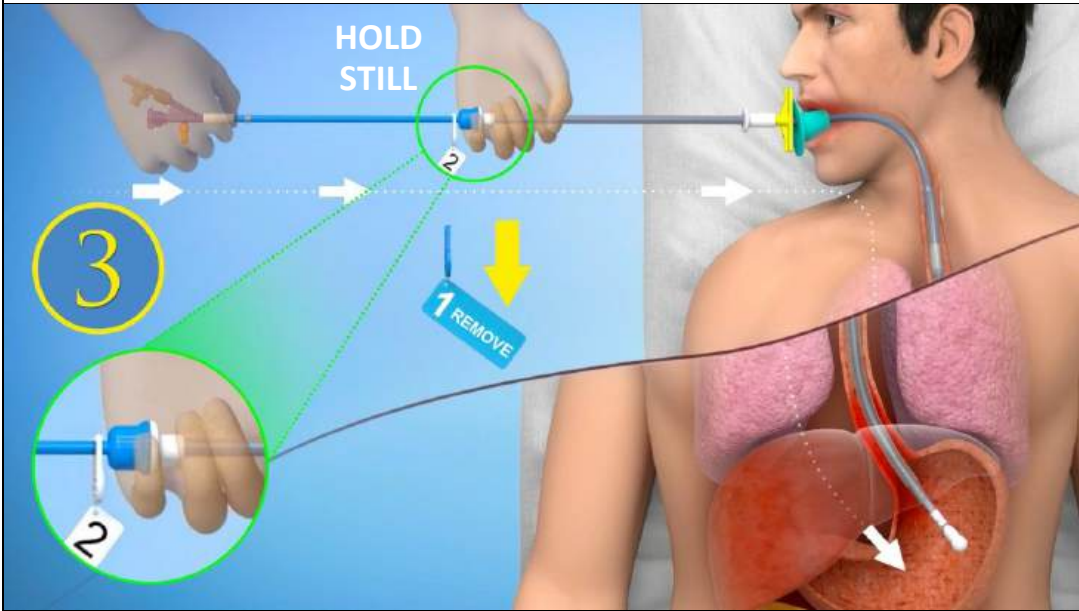


3 Remove the blue lock no. 1 and push forward the pusher handle.

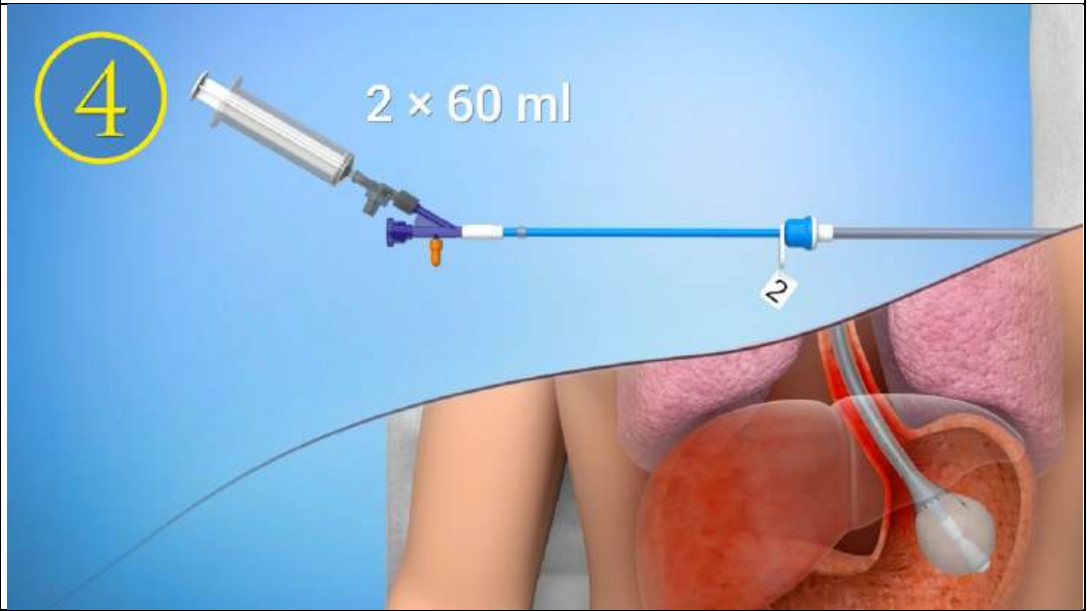


4 Inflate the balloon.

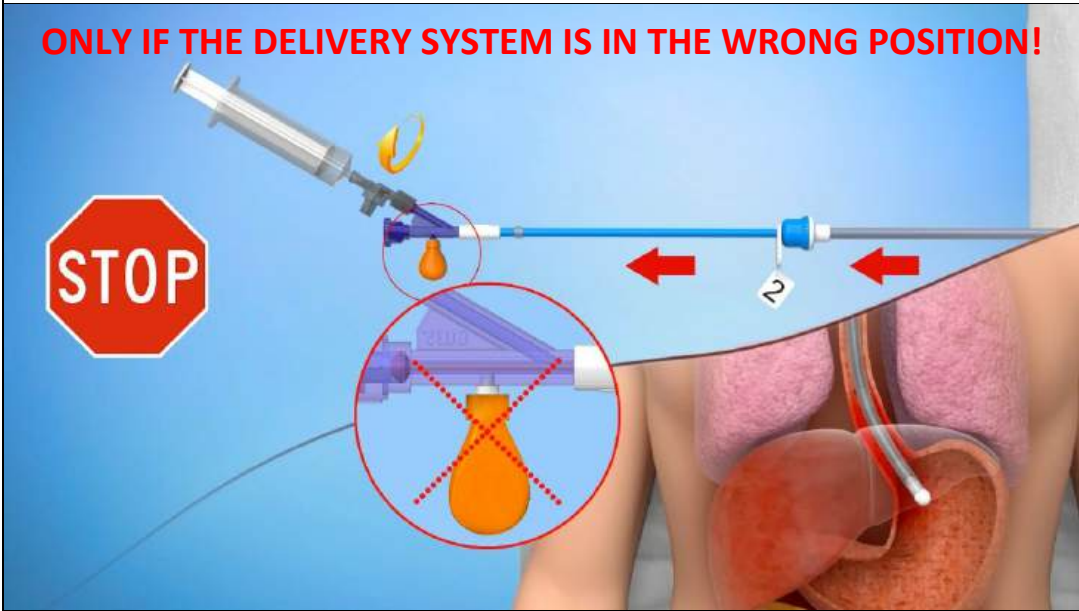


Unscrew the balloon valve and remove the delivery system.

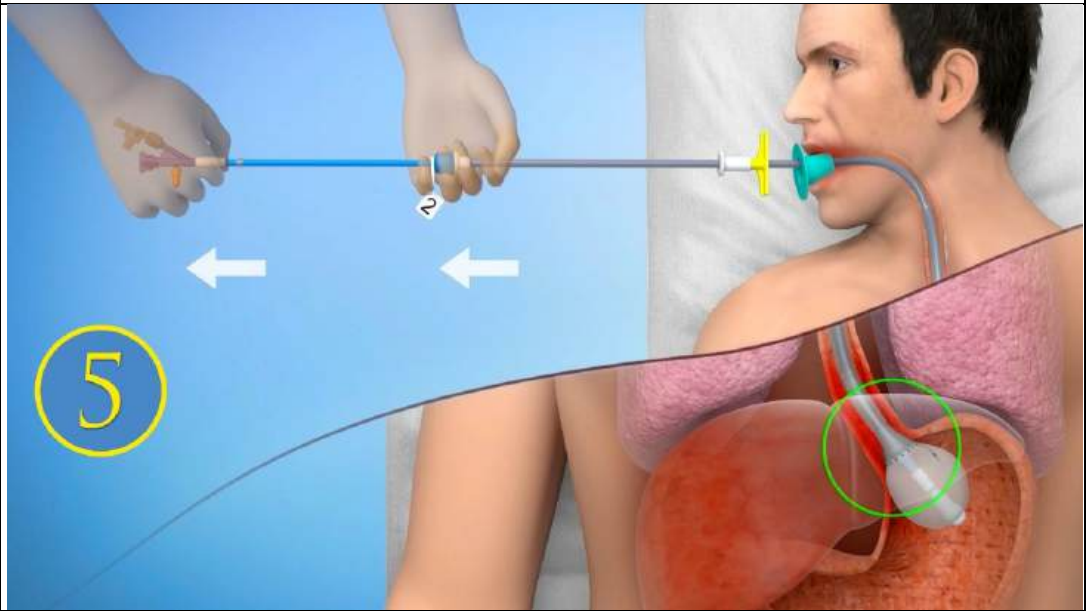

5 Pull back the whole delivery system to secure the balloon.

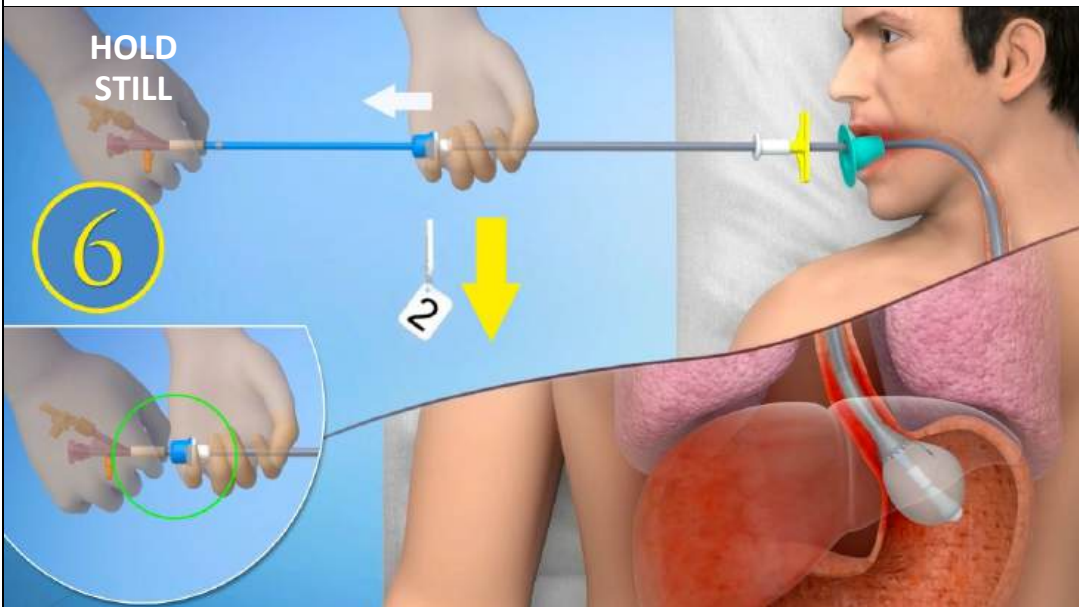

6 Remove the white lock no. 2 and pull back the sheath handle over the safety valve.

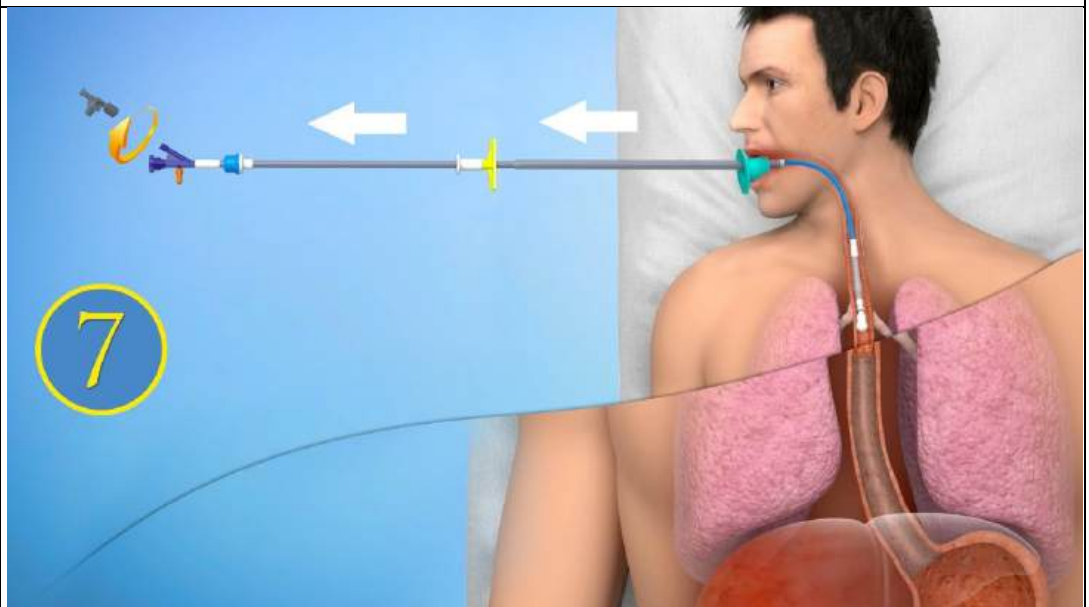

7 Unscrew the balloon valve and remove the delivery system.



Click here to view 3D animation

Issue date: January 2021

Proudly distributed by



1300 00 ENDO
 info@endotx.com.au
 endotx.com.au