



Balloon Extractor™ - Instructions for Use


1) Open package and assemble Balloon Catheter and Syringe

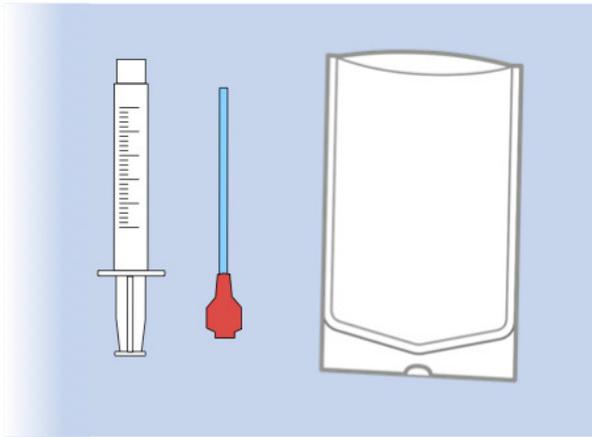


2) Insert past foreign body with deflated balloon

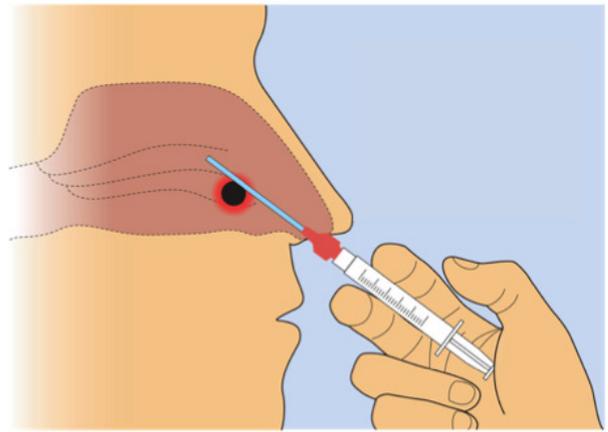

3) Inflate balloon till pressure of nasal cavity is felt

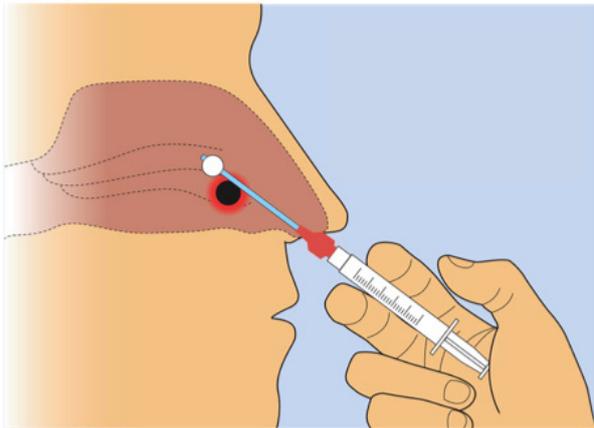

4) Extract foreign body by withdrawing balloon

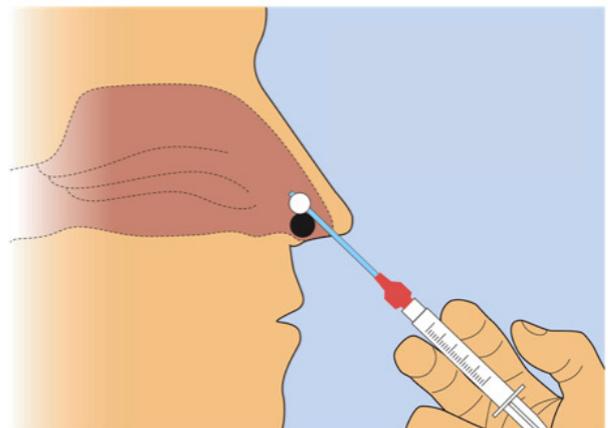


Contraindications

- 1) Do not use if the device appears damaged
- 2) Do not use if bleeding is evident in the nasal canal
- 3) Do not use if there is not adequate space to inflate the balloon
- 4) Not intended for use in ear canal

Specifications



—

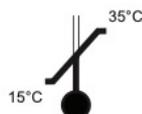

Single use only.

Proudly distributed by



- 1300 00 ENDO
- info@endotx.com.au
- endotx.com.au
- Suite 301, 2 Banfield Road
Macquarie Park NSW 2113